
GGI MapQuant Dx™ Process


Feasibility Study
IJB, Brussels 26/10/2009.

MapQuant Dx™ Process

A 3-step process

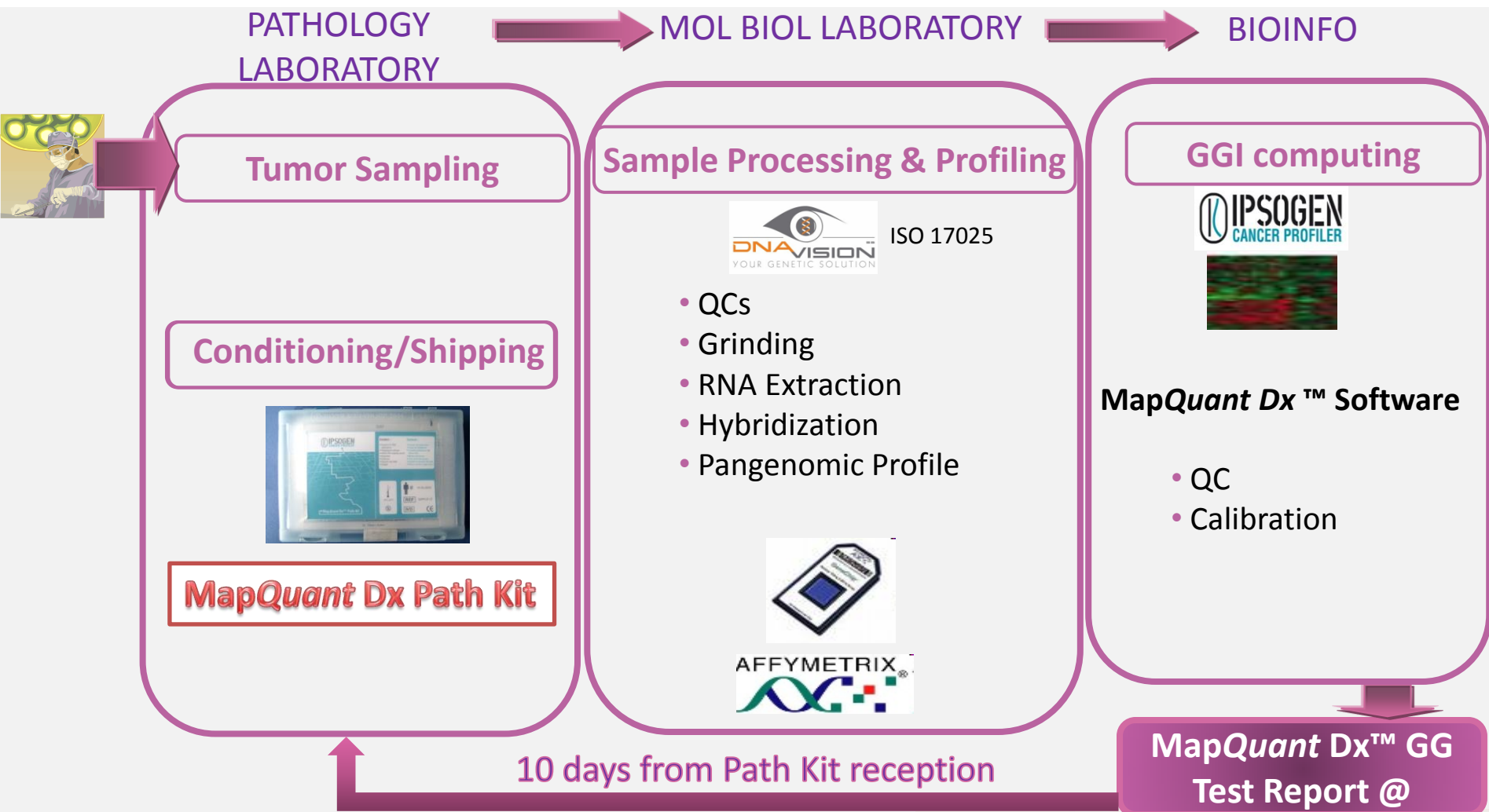Tumour Sampling Process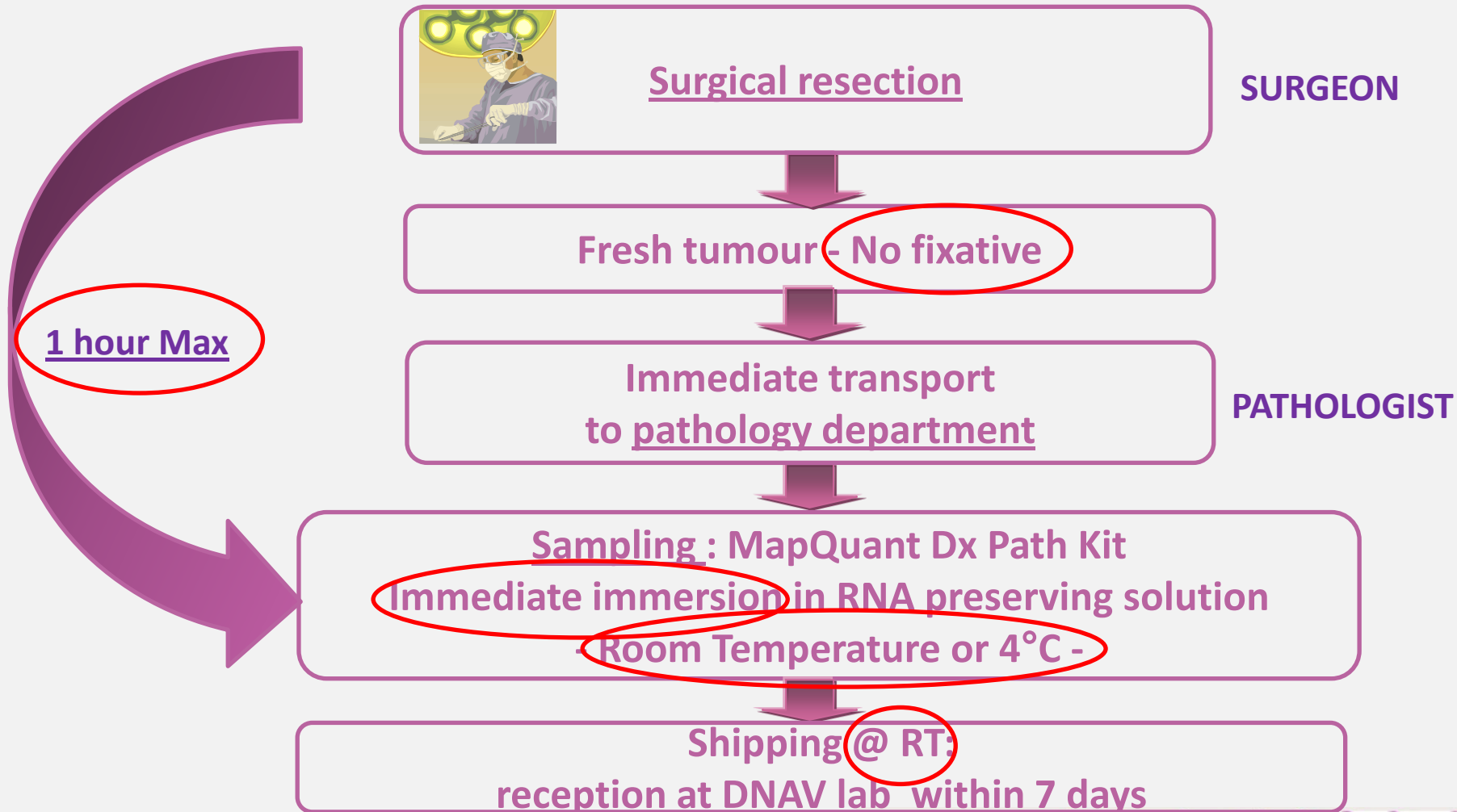

Tumour Sampling from fresh tissue and Storage

MapQuant Dx™ Path Kit: tissue sample collection & RNA preservation during storage and transport

PATHOLOGY LABORATORY


Sampling

- Surgical biopsy
- representative sample
- Ø 3 mm min.
- **Fresh**

Immediate immersion in
preservative solution
(**1 hour max** after resection)

Storage

- **never freeze**
- **+4°C or RT**

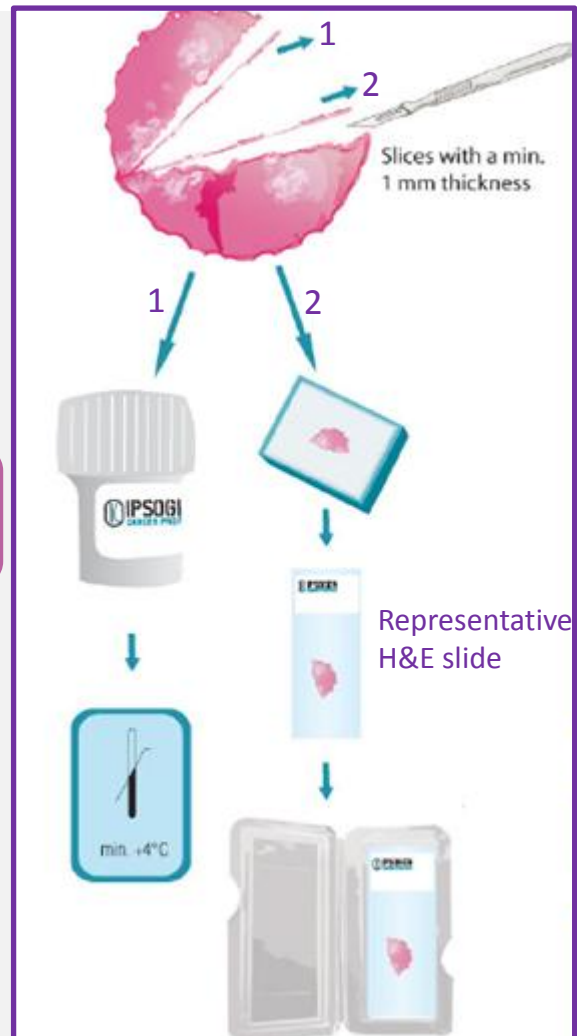Tumour Sampling from frozen tissue and Storage

MapQuant Dx™ Path Kit: tissue sample collection & RNA preservation during storage and transport

Nerver defrost

Frozen tissue

200µm (20µm microtome slices x10)

Immediate immersion in
preservative solution
(1 hour max after resection)

Storage

- never freeze
- +4°C or RT

MapQuant Dx Path Kit preparation for shipping

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Once completed, place this form, slide mailer and bag containing tube & absorbent in the outer blue box. Close the box and seal it with the security seal label.

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date and signature

- Specimen in RNA preservative solution
- H&E slide

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_____ date and signature

MapQuant Dx Genomic Grade Test Report

- Test report sent by e-mail and FAX:

MapQuant Dx™
Support +33(0)491 293090

GENOMIC GRADE REPORT


Institution (Prescriber):	Contact person:
Address:	Position:
Service:	Phone:
Specimen (Prescriber):	Fax:
	Email:
	Specimen ID (Ipsogen): PK-06-26153

TEST DESCRIPTION

MapQuant Dx™ Genomic Grade is an in vitro diagnostic test which quantifies the Genomic Grade index (GGi) from qualified Affymetrix GeneChip® microarray genomic profile obtained in an Ipsogen-certified partner laboratory. The expression levels of the 97 mRNAs which best discriminate grade 3 from grade 1 tumors are calibrated, averaged, and scaled using high-quality reference breast cancer profiles. Profiles are then classified as grade 1 or grade 3 according to their GGi value.

References:
1. Sotiriou et al. Gene expression profiling in breast cancer: understanding the molecular basis of histologic grade to improve prognosis. *J Natl Cancer Inst*. 2006; 98(4):262-72.
2. Liat et al. Definition of clinically distinct molecular subtypes in estrogen receptor positive breast cancer through genomic grade. *J Clin Oncol* 2007; 25(16):1233-45.

RESULT: GRADE ③

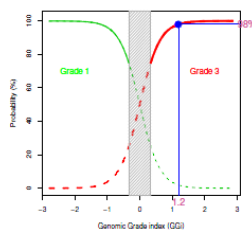✓ The genomic grade index is 1.2; which corresponds to a grade of 3 (with a probability of 98 %)

QUALITY CONTROLS

RNA	passed	✓
Genomic Profile	passed	✓

Notes

Name: _____ Position: _____
Date and Signature

IPSOGEN SA
Luminy Research Expression, Case 923, 143 avenue de Luminy
13288 Marseille cedex 9, France
Tel. +33(0)491 293090 Fax. +33(0)491 293099
Email: support@ipsogen.com Web: www.ipsogen.com

IPSOGEN SA
Luminy Research Expression, Case 923, 143 avenue de Luminy
13288 Marseille cedex 9, France
Tel. +33(0)491 293090 Fax. +33(0)491 293099
Email: support@ipsogen.com Web: www.ipsogen.com

MapQuant Dx™ is an in vitro diagnostic testing solution combining the use of MapQuant Dx™ Path XL™ (28-mers), with an ISO 15189:2013 certified lab. Service performed on Affymetrix® platform. Genomic indexes are computed through a Q&A compliant software. MapQuant Dx™ has been developed under the French lab Support Program of the French Health Products Safety Agency (ANSM). MapQuant Dx™ is a trademark of Ipsogen SA.